

ExEm Foam-Kit

Tubal patency test with clear results

New



Hysterosalpingo “Foam” Sonography (HyFoSy)

The ExEm Foam Kit enables easy recognition of tubal patency when performing Hysterosalpingo Foam Sonography.

The ExEm Foam-Kit gives a bright visualization of fallopian tubes and the uterine cavity contour.

Eliminates the need for X-ray

The technique is optimized for checking the patency of the fallopian tubes in infertility patients but the technique also enables verification of correct placement of the micro implants after sterilization procedure.

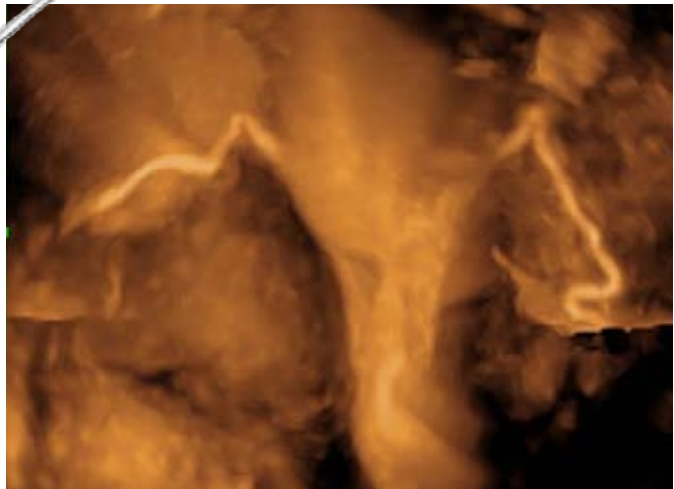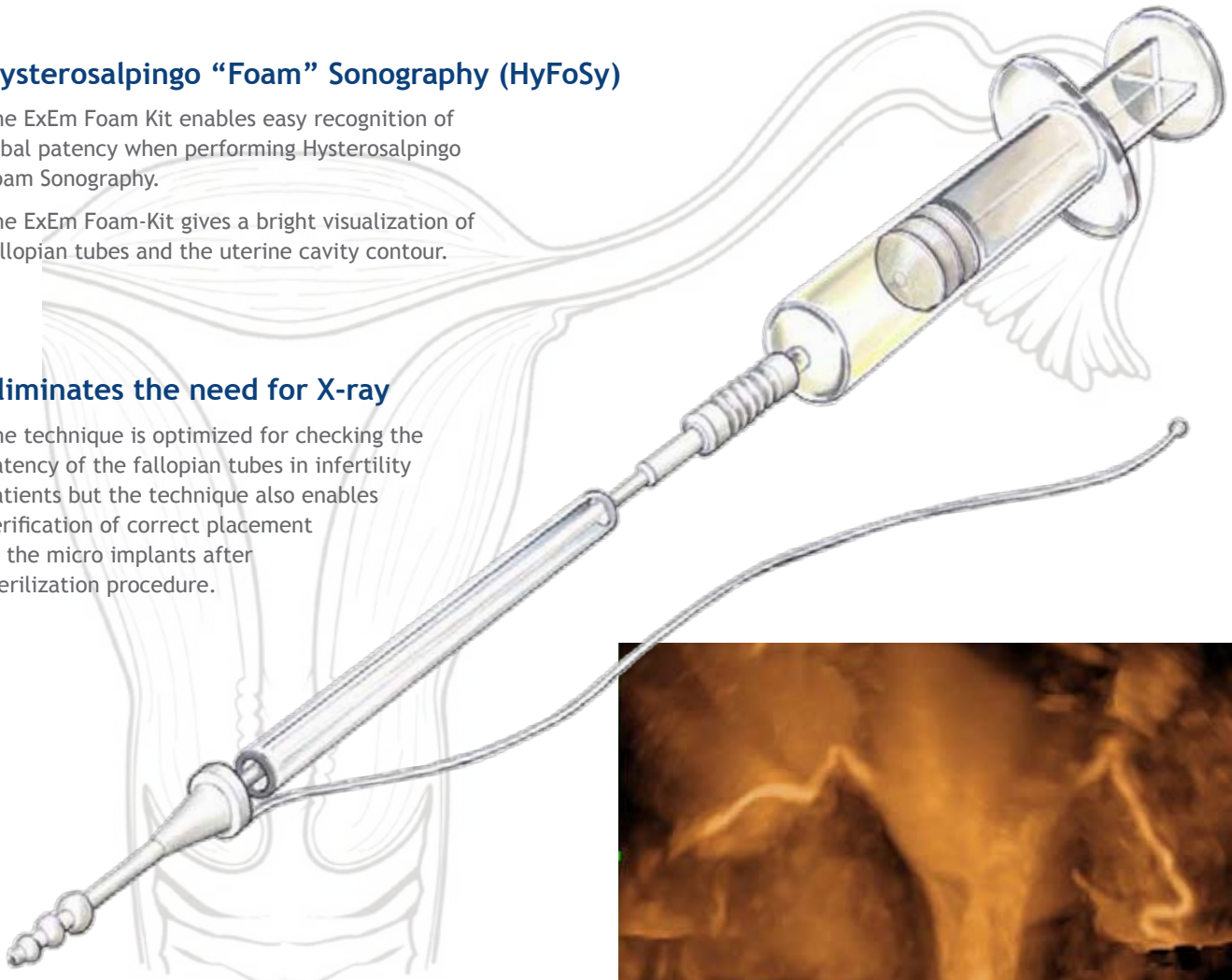

Simple, harmless and low cost

The ExEm® gel is a neutral gel which is mixed with purified water to create a foam with microbubbles. The foam will cause a fast and simple extension of the walls of the fallopian tubes. The foam that flows into the abdominal cavity will be processed by the human body. The remaining foam will be resorbed within 24 hours just like air remaining in the abdominal cavity after abdominal surgery.

During HyFoSy the fallopian tubes will be reflected on the ultrasound monitor. The tubes and a “spill” into the abdomen will be visible for a short period of time. If not, the passage of one or both of the fallopian tubes is disturbed.



Ref	Product	Package
13045	ExEm Foam Kit (incl. Gis catheter)	25 units/box
13046	ExEm Foam Kit (incl. Gis catheter)	10 units/box
13042	ExEm Foam Kit (excl. Gis catheter)	25 units/box
13044	GIS catheter - M969 (multipara)	25 units/box
13043	GIS catheter - N970 (nulipara)	25 units/box